

FAST TRACK DELIVERY
of large therapy sheaths
with a single solution


SEAMLESS
LARGE DILATOR

to optimize tissue dilation
for large sheath cases


3-IN-1
RF WIRE

for exchangeless delivery
of therapy sheaths


OPTIMIZED TISSUE DILATION
may be required for:

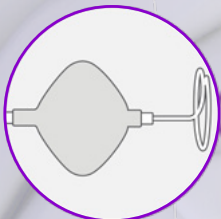

Cryoballoon
Ablation

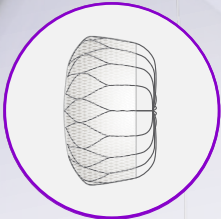

Left Atrial
Appendage
Closure

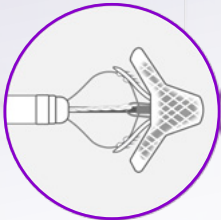

Mitral Valve
Repair



VersaCross™
RF Wire &
Dilator

INSERT

1

1

Guidewire
& sheath

2

POSITION
on fossa

2

RETRACT
guidewire

3

CONFIRM
position

3

INSERT
needle

4

TRANSSEPTAL
access

4

RETRACT
needle

5

RETRACT
dilator/sheath

5

6

DELIVER
therapy device

6

1. Choose your VersaCross™ RF Wire

Pigtail




J-Tip



Length: 180 cm, 230 cm
Diameter: 0.035"

2. Choose your VersaCross™
Large Access Transseptal Dilator

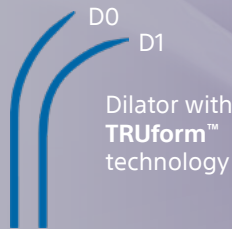

Inner Diameter: 0.035"
Outer Diameter: 12.5F
Useable Length: 67 cm, 85 cm

The VersaCross™ Large Access Solution
comes with a single-use connector cable
and mechanical guidewire

All trademarks are property of their respective owners. Patents Pending and/or issued. Caution: The law restricts these devices to sale by or on the order of a physician. Indications, Contraindications, Warnings, and Instructions For Use can be found in the product labelling supplied with each device or at www.baylismedical.com.

Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

Boston Scientific is a Global Company. Please note that model numbers, indications, contraindications, warnings and specifications may differ depending on geographic region. Not all information displayed in this brochure may be licensed in accordance with Canadian law. Please contact your Boston Scientific representative for local labeling, product specifications and licensed model numbers.

Boston
Scientific
Advancing science for life™

Baylis Medical Company Inc.
5959 Trans-Canada Highway
Montreal, QC Canada H4T 1A1

www.baylismedical.com
info@baylismedical.com

General Inquiries
(514) 488-9801

© 2023 Boston Scientific
Corporation or its affiliates.
All rights reserved.
EP-1548906-AA



VersaCross™
Large Access Solution

ACCESS
DILATE &
DELIVER

large therapy sheaths with a
single, exchangeless* solution

*VersaCross™ RF Wire can be used, without exchanges, as a guidewire, as a transseptal puncture device or as an exchange rail for delivering therapy sheaths.

ACCESS

with precision


DILATE

to optimize delivery of
large therapy sheaths


DELIVER

large therapy sheaths effortlessly


EXCHANGELESS 3-in-1 RF Wire

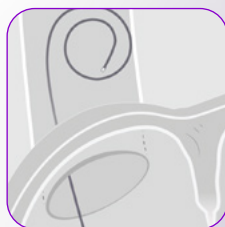


1 Start at the SVC

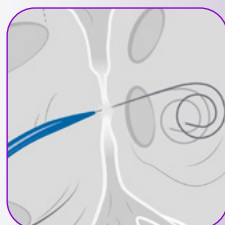

2 RF transseptal puncture

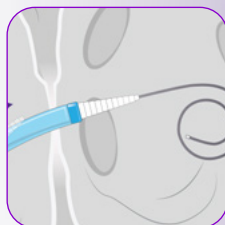

3 Supportive exchange rail

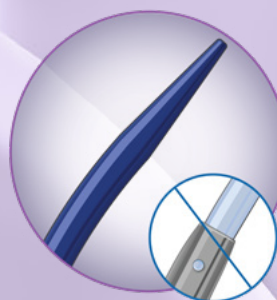

SINGLE SEAMLESS

dilator for controlled
crossing into the
left atrium



12.5F DILATION

at the femoral vein
and septum to deliver
large sheaths with ease

INSTANTLY DEPLOY 0.035" WIRE

to deliver therapy sheaths
with confidence

DELIVER THERAPY ON TARGET



Precision RF
Puncture Technology



TRUform™
Shapeable Technology



OMNIviz™ Technology

Reliably locate your solution
on fluoroscopy, ultrasound
and visibly with positional markers