



Highlights from:

Gregory K. Feld, Jay Tiongson, and Ganiyu Oshodi

Feld et al., J Interv Card Electrophysiol, Jan 2011 DOI: 10.1007/s10840-010-9531-3


Particle formation and risk of embolization during transseptal catheterization: Comparison of standard transseptal needles and a new radiofrequency transseptal needle

INTRODUCTION


- ▶ This study examined the risk of particle formation (due to skiving) during transseptal procedures. A standard needle, a reverse-bevel needle, and a radiofrequency needle were compared.

DISCUSSION AND CONCLUSIONS


- ▶ The standard needle generated clinically relevant, visible particles when used without the stylet. Use of the stylet reduced the amount of visible particles generated but did not eliminate the problem.
- ▶ Use of the reverse-bevel needle reduced the amount of visible particles generated but did not eliminate the problem.
- ▶ Use of the radiofrequency needle eliminated the problem of visible particles.

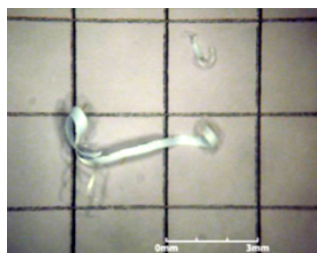


Standard needle without stylet

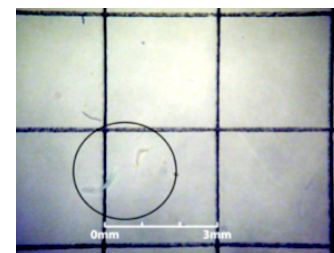

Standard needle with stylet

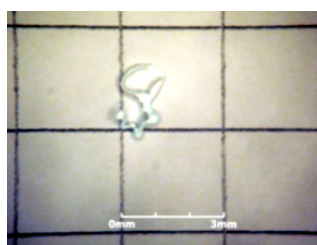

Reverse-bevel needle

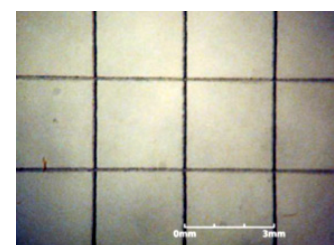

NRG™ Needle

All trademarks are property of their respective owners. Patents Pending and/or issued. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, Contraindications, Warnings, and Instructions For Use can be found in the product labelling supplied with each device or at www.baylismedical.com.

Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries.
This material not intended for use in France.



**Boston
Scientific**
Advancing science for life™

© 2023 Boston Scientific Corporation
or its affiliates. All rights reserved.
EP-1613909-AA