## Instructions for Use SupraCross® RF Wire

|                 |    |
|-----------------|----|
| English .....   | 1  |
| Français.....   | 3  |
| Deutsch .....   | 4  |
| Nederlands..... | 6  |
| Italiano.....   | 8  |
| Español .....   | 9  |
| Português.....  | 11 |

Baylis Medical Company Inc.  
5959 Trans-Canada Highway  
Montreal, Quebec, Canada, H4T 1A1  
Tel: (514) 488-9801/ (800) 850-9801  
Fax: (514) 488-7209  
www.baylismedical.com

**EU Authorized Representative:**  
Quality First International  
20 Eversley Road, Bexhill-on-Sea, East Sussex  
TN40 1HE, United Kingdom  
Tel: +44-(20)-8-522-1937 Fax: +44-(20)-8-522-1937

© Copyright Baylis Medical Company Inc., 2016-2018

The Baylis Medical Company logo and SupraCross are trademarks and/or registered trademarks of Baylis Medical Company Inc. in the USA and/or other countries. Patents pending and/or issued.

**English**  
Carefully read all instructions prior to use. Observe all contraindications, warnings and precautions noted in these instructions. Failure to do so may result in patient complications.  
**Caution:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

### I. DEVICE DESCRIPTION

The *SupraCross* RF Wire is packaged with a Baylis Connector Cable. The *SupraCross* RF Wire must be used with an approved Baylis Radiofrequency Puncture Generator (Baylis RF Generator) and Baylis Connector Cable (Connector Cable).  
The *SupraCross* RF Wire delivers radiofrequency (RF) power in a monopolar mode between its distal electrode and a commercially available external Disposable Indifferent (Dispersive) Patch (DIP) Electrode, which is in compliance with IEC 60601-2-2 standards. The Connector Cable connects the Baylis RF Generator to the *SupraCross* RF Wire. This Connector Cable enables RF power to be delivered from the Baylis RF Generator to a *SupraCross* RF Wire.  
Detailed information concerning the Baylis RF Generator is contained in a separate manual that accompanies the Generator (entitled "Baylis Radiofrequency Puncture Generator Instructions for Use"). Baylis RF Generators compatible with the *SupraCross* RF Wire include the RFP-100A. The dimensions of the *SupraCross* RF Wire and the Baylis Connector Cable can be found on the device label. The insulation on the body of the *SupraCross* RF Wire facilitates smooth advancement of the device and provides electrical insulation. The floppy distal portion of the *SupraCross* RF Wire has a small curve and the active tip is rounded to be atraumatic to cardiac tissue unless RF energy is applied. A marker coil is positioned on the curve for visualization under fluoroscopy. The main body of the *SupraCross* RF Wire provides a stiff rail for advancing ancillary devices into the left atrium following creation of an atrial septal defect. The floppy proximal portion of the *SupraCross* RF Wire has a large circular curve and the proximal end is bare metal to connect with the included Connector Cable. The other end of the included Connector Cable connects to the Baylis RF Generator.

### II. INDICATIONS FOR USE

The *SupraCross* RF Wire is indicated for creation of an atrial septal defect in the heart.

### III. CONTRAINDICATIONS

The *SupraCross* RF Wire is not recommended for use with any conditions that do not require the creation of an atrial septal defect.  
The Baylis Connector Cable, packaged with the *SupraCross* RF Wire, is not recommended for use with any other RF Generator or any other device.

### IV. WARNINGS

- Only physicians with a thorough understanding of angiography and percutaneous interventional procedures should use this device. It is recommended that physicians avail themselves of pre-clinical training, a review of pertinent literature and other appropriate education before attempting new interventional procedures.
- The *SupraCross* RF Wire and Baylis Connector Cable (Connector Cable) are supplied STERILE using an ethylene oxide process. Do not use if the package is damaged.
- Laboratory staff and patients can undergo significant x-ray exposure during radiofrequency puncture procedures due to the continuous usage of fluoroscopic imaging. This exposure can result in acute radiation injury as well as increased risk for somatic and genetic effects. Therefore, adequate measures must be taken to minimize this exposure.
- The *SupraCross* RF Wire and Connector Cable are intended for single patient use only. Do not attempt to sterilize and reuse either device. Reuse can cause the patient injury and/or the communication of infectious disease(s) from one patient to another. Failure to do so may result in patient complications.
- The *SupraCross* RF Wire must be used with the Connector Cable provided. Attempts to use it with other connector cables can result in electrocution of the patient and/or operator.
- The Connector Cable must only be used with Baylis Radiofrequency Puncture Generator (Baylis RF Generator) and the included *SupraCross* RF Wire. Attempts to use it with other RF Generators and devices can result in electrocution of the patient and/or operator.
- The *SupraCross* RF Wire must be used with 0.035" Compatible Transseptal Sheath and Dilator devices.

- The active tip and distal curve of the *SupraCross* RF Wire are fragile. Be careful not to damage the tip or the distal curve while handling the *SupraCross* RF Wire. If the tip or the distal curve becomes damaged, discard the *SupraCross* RF Wire immediately.
- If the active tip of the *SupraCross* RF Wire becomes bent at any time during its use, dispose of the *SupraCross* RF Wire immediately. Do not attempt to straighten the active tip.
- The *SupraCross* RF Wire is not intended for use with neonatal patients (less than one month of age). Do not attempt to treat neonatal patients with the *SupraCross* RF Wire.

### V. PRECAUTIONS

- Do not attempt to use the *SupraCross* RF Wire and Baylis Connector Cable (Connector Cable) or ancillary equipment before thoroughly reading the accompanying Instructions for Use.
- Radiofrequency puncture procedures should be performed only by physicians thoroughly trained in the techniques of radiofrequency powered puncture in a fully equipped catheterization laboratory.
- The sterile packaging should be visually inspected prior to use to detect any compromise. Ensure that the packaging has not been damaged. Do not use the equipment if the packaging has been compromised.
- Visually inspect the *SupraCross* RF Wire and Connector Cable prior to use to ensure there is no cracking or damage to the insulating material. Do not use the wire or the cable if there is any damage.
- Do not use the *SupraCross* RF Wire and/or Connector Cable after the USE BY date indicated on the label.
- The *SupraCross* RF Wire and Connector Cable is intended for use with only those devices listed in Section VIII, Equipment Required.
- Read and follow the manufacturer's instructions for use of the Disposable Indifferent (Dispersive) Patch (DIP) electrode. Always use DIP electrodes that meet or exceed IEC 60601-2-2 requirements.
- Placement of the dispersive electrode on the thigh could be associated with higher impedance.
- In order to prevent the risk of ignition, ensure that flammable materials are not present in the room during RF power application.
- Take precautions to limit the effects that the electromagnetic interference (EMI) produced by the Baylis Radiofrequency Puncture Generator (Baylis RF Generator) may have on the performance of other equipment. Check the compatibility and safety of combinations of other physiological monitoring and electrical apparatus to be used on the patient in addition to the Baylis RF Generator.
- Adequate filtering must be used to allow continuous monitoring of the surface electrocardiogram (ECG) during radiofrequency power applications.
- Do not attempt to insert and use the proximal end of the *SupraCross* RF Wire as the active tip.
- Do not attempt to insert or retract the *SupraCross* RF Wire through a metal cannula or a percutaneous needle.
- Do not bend the *SupraCross* RF Wire or the Connector Cable. Excessive bending or kinking of the wire shaft, distal curve, or proximal curve may damage the integrity of the *SupraCross* RF Wire and may cause patient injury. Care must be taken when handling the RF Wire and Connector Cable.
- Careful manipulation of the *SupraCross* RF Wire must be performed to avoid vessel trauma. *SupraCross* RF Wire and Dilator advancement should be done under imaging guidance. If resistance is encountered, DO NOT use excessive force to advance or withdraw the *SupraCross* RF Wire or Dilator.
- Do not attempt to deliver radiofrequency energy until the active tip of the *SupraCross* RF Wire is confirmed to be in good contact with the target tissue.
- It is recommended not to exceed five (5) RF power applications per *SupraCross* RF Wire.
- Never disconnect the Connector Cable from the Baylis RF Generator while the Generator is delivering RF power.
- Never disconnect the Connector Cable from the Baylis RF Generator by pulling on the cable. Failure to disconnect the cable properly may result in damage to the cable.
- Do not twist the Connector Cable while inserting or removing it from the Isolated Patient Connector on the Baylis RF Generator. Twisting the cable may result in damage to the pin connectors.
- The Baylis RF Generator is capable of delivering significant electrical power. Patient or operator injury can result from improper handling of the *SupraCross* RF Wire and/or DIP electrode, particularly when operating the device.
- During power delivery, the patient should not be allowed to come in contact with ground metal surfaces.
- Apparent low power output or failure of the equipment to function properly at normal settings may indicate faulty application of the DIP electrode, failure to an electrical lead, or poor tissue contact at the active tip. Check for obvious equipment defects or misapplication. Attempt to better position the active tip of the *SupraCross* RF Wire against the atrial septum. Only increase the power if low power output persists.
- Baylis Medical Company relies on the physician to determine, assess and communicate to each individual patient all foreseeable risks of the Baylis Medical Radiofrequency Puncture System.

### VI. PRODUCT SPECIFICATIONS

| Product          | <i>SupraCross</i> RF Wire              | Product             | RFP 100A Connector Cable |
|------------------|--|---------------------|--------------------------|
| Useable Length   | 180cm                                  | Useable Length      | 10 feet/3m               |
| Outside Diameter | 0.035" / 0.89mm                        | Generator Connector | 4-pin (1+pin)            |
| Marker Coil      | 1 x 30mm Coil<br>Proximal of Electrode | Device Connector    | Push Button              |

### VII. ADVERSE EVENTS

Adverse events that may occur while creating an atrial septal defect include:

|                     |                                      |                               |
|---------------------|--------------------------------------|-------------------------------|
| Tamponade           | Sepsis/Infection                     | Thromboembolic episodes       |
| Vessel perforation  | Atrial Fibrillation                  | Myocardial Infarction         |
| Vessel spasm        | Sustained arrhythmias                | Atrial Flutter                |
| Hemorrhage          | Vascular thrombosis                  | Perforation of the myocardium |
| Hematoma            | Allergic reaction to contrast medium | Ventricular Tachycardia       |
| Pain and Tenderness | Arteriovenous fistula                |                               |

### VIII. EQUIPMENT REQUIRED

Radiofrequency transeptal procedures should be performed in a specialized clinical setting equipped with a fluoroscopy unit, radiographic table, echocardiography imaging, physiologic recorder, emergency equipment and instrumentation for gaining vascular access. Ancillary materials required to perform this procedure include:

- Baylis Radiofrequency Puncture Generator (RFP-100A)
- 0.035" Compatible Transseptal Sheath/Dilator kit
- Disposable Indifferent (dispersive) Patch (DIP) electrode must meet or exceed IEC 60601-2-2 requirements for electrosurgical electrodes

### IX. INSPECTION PRIOR TO USE

Prior to performing the procedure, the individual components including the Baylis Radiofrequency Puncture Generator, *SupraCross* RF Wire and the Baylis Connector Cable (Connector Cable) should be carefully examined for damage or defects, as should all equipment used in the procedure. Do not use defective equipment. Do not reuse the RF Wire and/or Connector Cable.


### X. DIRECTIONS FOR USE

- All instructions for equipment required should be carefully read, understood, and followed. Failure to do so may result in complications.
- The *SupraCross* RF Wire and Baylis Connector Cable (Connector Cable) are supplied sterile. Use aseptic technique when opening the package and handling the product in the sterile field.
- Connect the generator connector end of the Connector Cable to the isolated patient connector port on the Baylis Radiofrequency Puncture RF Generator (Baylis RF Generator) as per the Baylis RF Generator Instructions for Use. Gently line up the connector pins with the socket and push in until

the connector fits firmly into the socket. Any attempt to connect the cable otherwise will damage the pins on the connector.

- Do not use excessive force in connecting the Connector Cable to the Baylis RF Generator. Use of excessive force may result in damage to the connector pins.
- Thoroughly flush the Transseptal Dilator (not supplied).
- Perform a standard vein puncture at the desired access site using an access needle (not supplied).
- A Transseptal Sheath and Dilator are usually inserted through the access site and are then advanced over a guidewire to be positioned into the superior vena cava (SVC) under fluoroscopic guidance.
- Remove the guidewire.
- Using the provided tip straightener, straighten the distal curve of the *SupraCross* RF Wire.
- Insert the tip of the *SupraCross* RF Wire into the dilator hub and advance wire through the sheath/dilator set until the wire tip is just within the dilator tip.
- Firmly grasp the catheter connector end of the Connector Cable in one hand. Using your thumb depress the red button on the top of the connector. Slowly insert the proximal end of the *SupraCross* RF Wire into the opening of the catheter connector. Once the exposed portion of the proximal end of the device is no longer visible release the red button on the connector. Gently tug on the device to ensure that you have a secure connection.
- Position the wire/sheath/dilator assembly in the right atrium against the fossa ovalis under fluoroscopic guidance using standard technique. Echocardiographic guidance is also recommended.
- Apply pressure to the dilator to tent the septum at the fossa ovalis.
- Advance the *SupraCross* RF Wire so that the active tip is engaging the septum at the fossa ovalis, but still within the dilator.
- Once appropriate positioning has been achieved, deliver RF power via the Baylis RF Generator to the active tip. This results in puncture of the targeted cardiac tissue. Please refer to the Baylis RF Generator Instructions for Use for the correct operation of the generator.
- Apply firm pressure to the *SupraCross* RF Wire during the application of RF energy to successfully advance *SupraCross* RF Wire through the tissue.
- NOTE: For Baylis RF Generator units, use the lowest appropriate RF settings to achieve the desired puncture.**
  - For RFP-100A: An initial RF setting between one (1) second on "PULSE" mode to two (2) seconds on "CONSTANT" mode has been shown to be sufficient for successful puncture.
- Radiofrequency power delivery can be terminated by pressing the RF ON/OFF button on the Generator if the timer has not expired.
- Entry into the left atrium can be confirmed by monitoring the *SupraCross* RF Wire under fluoroscopy. Echocardiographic guidance is also recommended.
- If septal puncture is not successful after five (5) RF power applications, it is advised that the user utilize an alternate method for the procedure.
- Once the puncture is successfully completed, the *SupraCross* RF Wire should be mechanically advanced without any RF power. Positioning in the left atrium is sufficient when the full distal curve has crossed the septum and is observed under fluoroscopy in the left atrium. Echocardiographic guidance is also recommended.
- The Transseptal Dilator can then be advanced over the wire to enlarge the puncture.
- To disconnect the *SupraCross* RF Wire from the Connector Cable, depress the red button on the catheter connector and gently remove the proximal end of the RF Wire from the connector cable.
- To disconnect the Connector Cable from the Baylis RF Generator, grasp the connector firmly and gently pull it straight out of the socket.
- Retract the *SupraCross* RF Wire slowly through the Transseptal Sheath/Dilator set.

### Connections

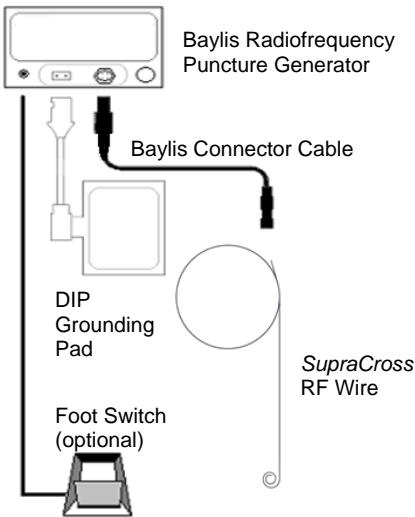### XI. CLEANING AND STERILIZATION INSTRUCTIONS

The *SupraCross* RF Wire and Baylis Connector Cable is intended for single use only. Do not clean or re-sterilize the *SupraCross* RF Wire and/or Baylis Connector Cable.

### XII. TROUBLESHOOTING

The following table is provided to assist the user in diagnosing potential problems.

| PROBLEM   | COMMENTS   | TROUBLESHOOTING  |
|---|--|--|
| Baylis Connector Cable does not fit into the Isolated Patient Connector on the front panel of the generator | The connectors are designed to connect in a specific way for safety reasons. If the connector "keys" are out of line, the connectors won't fit together. | Check that the connector keys are lined up in the proper orientation.  |
| Generator Error Messages  | In order to successfully perforate tissue using radiofrequency energy, all devices must be properly connected and in good working order.                 | Ensure that all connections are made: <ul style="list-style-type: none"> <li>- <i>SupraCross</i> RF Wire to Connector Cable</li> <li>- Connector Cable to Baylis RF Generator</li> <li>- Baylis RF Generator to power outlet</li> <li>- Baylis RF Generator to grounding pad</li> </ul> Visually inspect the <i>SupraCross</i> RF Wire and Connector Cable for damage. Immediately discard any damaged equipment. If problem persists discontinue use. For error messages encountered while attempting radiofrequency puncture, refer to the Instructions for Use document that accompanies the Baylis RF Generator. |
| Wire breaks or kinks  | Breaks and kinks in the <i>SupraCross</i> RF Wire are a  | Discard immediately.   |

potential cause of patient injury.

### XIII. CUSTOMER SERVICE AND PRODUCT RETURN INFORMATION

If you have any problems with or questions about Baylis Medical Equipment contact our technical support personnel at the following address and/or phone number.

**Baylis Medical Company Inc.**  
5959 Trans-Canada Highway  
Montreal, Quebec, Canada, H4T 1A1  
Phone: (514) 488-9801 or (800) 850-9801  
Fax: (514) 488-7209  
www.baylismedical.com

### NOTES:

- In order to return products you must have a return authorization number before shipping the products back to Baylis Medical Company. Product Return Instructions will be provided to you at this time.
- Ensure that any product being returned to Baylis Medical has been cleaned, decontaminated and/or sterilized as indicated in the Product Return Instruction before returning it for warranted service. Baylis Medical will not accept any piece of used equipment that has not been properly cleaned or decontaminated as per the Product Return Instructions.

### XIV. LABELING AND SYMBOLS

|  |  |  |                                    |
|--|--|--|------------------------------------|
|  | Keep Dry   |  | Use By                             |
|  | Manufacturer   |  | Caution                            |
|  | Lot Number   |  | Keep Away from Sunlight            |
|  | Model Number   |  | Non-Pyrogenic                      |
|  | Do Not Resterilize   |  | Follow Instructions for Use        |
|  | Do Not Reuse   |  | Do Not Use if Packaging is Damaged |
|  | Sterile using ethylene oxide   |  |                                    |
|  | <b>Caution:</b> Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.  |  |                                    |
|  | <b>Only for EU member states:</b><br>Use of this symbol indicates that the product must be disposed of in a way that complies with local and national regulations. For questions regarding recycling of this device please contact your distributor. |  |                                    |

### XV. LIMITED WARRANTY – DISPOSABLES AND ACCESSORIES

Baylis Medical Company Inc. (BMC) warrants its Disposable and Accessory products against defects in materials and workmanship. BMC warrants that sterile products will remain sterile for a period of time as shown on the label as long as the original package remains intact. Under this Limited Warranty, if any covered product is proved to be defective in materials or workmanship, BMC will replace or repair, in its absolute and sole discretion, any such product, less any charges to BMC for transportation and labor costs incidental to inspection, removal or restocking of product. The length of the warranty is: (i) for the Disposable products, the shelf life of the product, and (ii) for the Accessory products, 90 days from shipment date.

This limited warranty applies only to new original factory delivered products that have been used for their normal and intended uses. BMC's Limited Warranty shall not apply to BMC products which have been resterilized, repaired, altered, or modified in any way and shall not apply to BMC products which have been improperly stored or improperly cleaned, installed, operated or maintained contrary to BMC's instructions.

#### DISCLAIMER AND LIMITATION OF LIABILITY

THE LIMITED WARRANTY ABOVE IS THE SOLE WARRANTY PROVIDED BY SELLER. SELLER DISCLAIMS ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE OR PURPOSE.

THE REMEDY SET FORTH HEREIN SHALL BE THE EXCLUSIVE REMEDY FOR ANY WARRANTY CLAIM, AND ADDITIONAL DAMAGES, INCLUDING CONSEQUENTIAL DAMAGES OR DAMAGES FOR BUSINESS INTERRUPTION OR LOSS OF PROFIT, REVENUE, MATERIALS, ANTICIPATED SAVINGS, DATA, CONTRACT, GOODWILL OR THE LIKE (WHETHER DIRECT OR INDIRECT IN NATURE) OR FOR ANY OTHER FORM OF INCIDENTAL, OR INDIRECT DAMAGES OF ANY KIND, SHALL NOT BE AVAILABLE. SELLER'S MAXIMUM CUMULATIVE LIABILITY RELATIVE TO ALL OTHER CLAIMS AND LIABILITIES, INCLUDING OBLIGATIONS UNDER ANY INDEMNITY, WHETHER OR NOT INSURED, WILL NOT EXCEED THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM OR LIABILITY. SELLER DISCLAIMS ALL LIABILITY RELATIVE TO GRATUITOUS INFORMATION OR ASSISTANCE PROVIDED BY, BUT NOT REQUIRED OF SELLER HEREUNDER. ANY ACTION AGAINST SELLER MUST BE BROUGHT WITHIN EIGHTEEN (18) MONTHS AFTER THE CAUSE OF ACTION ACCRUES. THESE DISCLAIMERS AND LIMITATIONS OF LIABILITY WILL APPLY REGARDLESS OF ANY OTHER CONTRARY PROVISION HEREOF AND REGARDLESS OF THE FORM OF ACTION, WHETHER IN CONTRACT, TORT (INCLUDING NEGLIGENCE AND STRICT LIABILITY) OR OTHERWISE, AND FURTHER WILL EXTEND TO THE BENEFIT OF SELLER'S VENDORS, APPOINTED DISTRIBUTORS AND OTHER AUTHORIZED RESELLERS AS THIRD-PARTY BENEFICIARIES. EACH PROVISION HEREOF WHICH PROVIDES FOR A LIMITATION OF LIABILITY, DISCLAIMER OF WARRANTY OR CONDITION OR EXCLUSION OF DAMAGES IS SEVERABLE AND INDEPENDENT OF ANY OTHER PROVISION AND IS TO BE ENFORCED AS SUCH.

IN ANY CLAIM OR LAWSUIT FOR DAMAGES ARISING FROM ALLEGED BREACH OF WARRANTY, BREACH OF CONTRACT, NEGLIGENCE, PRODUCT LIABILITY OR ANY OTHER LEGAL OR EQUITABLE THEORY, THE BUYER SPECIFICALLY AGREES THAT BMC SHALL NOT BE LIABLE FOR DAMAGES OR FOR LOSS OF PROFITS, WHETHER FROM BUYER OR BUYER'S CUSTOMERS. BMC'S LIABILITY SHALL BE LIMITED TO THE PURCHASE COST TO BUYER OF THE SPECIFIED GOODS SOLD BY BMC TO BUYER WHICH GIVE RISE TO THE CLAIM FOR LIABILITY.

No agent, employee or representative of Baylis Medical has the authority to bind the Company to any other warranty, affirmation or representation concerning the product.

This warranty is valid only to the original purchaser of Baylis Medical products directly from a Baylis Medical authorized agent. The original purchaser cannot transfer the warranty.

Use of any BMC product shall be deemed acceptance of the terms and conditions herein.

The warranty periods for Baylis Medical products are as follows:

|                     |                                |
|---------------------|--------------------------------|
| Disposable Products | The shelf life of the product  |
| Accessory Products  | 90 days from the shipment date |

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