

**ExpanSure<sup>®</sup>**  
Large Access Transseptal Dilator



**12.5F**  
Outer diameter

SEAMLESS ACCESS IN LARGE SHEATH CASES.™

# ExpanSure®


## Large Access Transseptal Dilator

### HIGHLIGHTS



### BENEFITS

#### 1 Optimizes tissue dilation for large sheath access

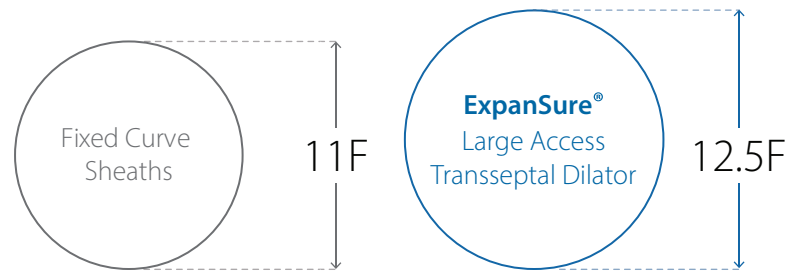


#### Large Outer Diameter (12.5F)


- **29%** larger dilation area than a standard fixed curve sheath

#### Facilitates Large Sheath Access

- Optimized dilation of septum may facilitate transseptal crossing required for left heart procedures, including:
  - Cryoballoon procedures
  - Left atrial appendage closure
  - Repeat procedures and/or fibrotic septa

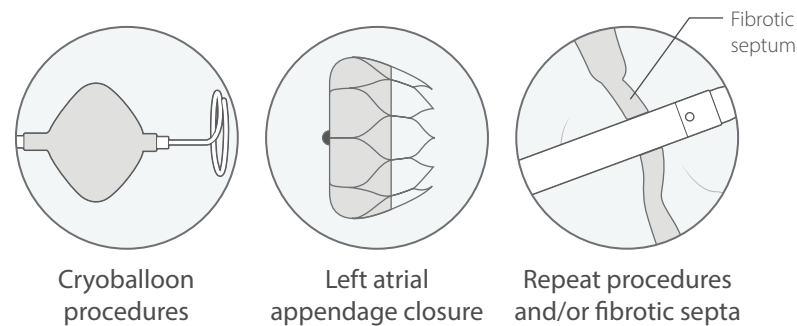


#### 2 Enhanced visibility to reliably locate your devices

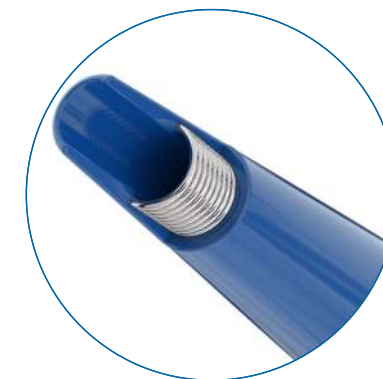

#### Enhanced Visibility


- Radiopaque coil for visualizing the distal tip of the dilator during manipulation on echo or fluoro

#### 3 Streamlined design for seamless crossing

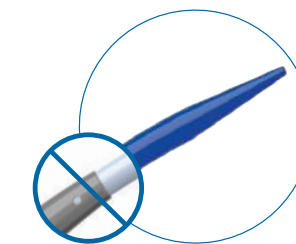

#### No Dilator-Sheath Transition

- Single device replaces fixed curve sheath and dilator
- Sleek profile for seamless advancement across septum

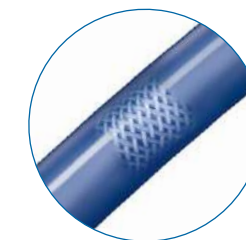


#### Control and Stability

- Braided composite design for stability and support during crossing


# INTEGRATES INTO EXISTING WORKFLOW

The **ExpanSure**® device replaces your traditional sheath and dilator for seamless integration into your transeptal procedure


Introduce guidewire into superior vena cava (SVC)

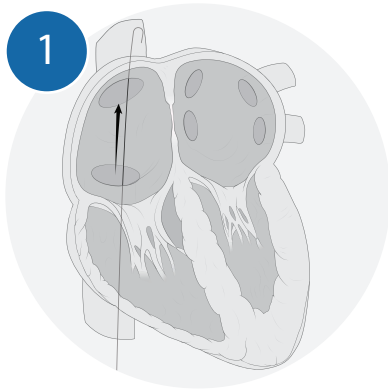


Introduce **ExpanSure**® device over guidewire

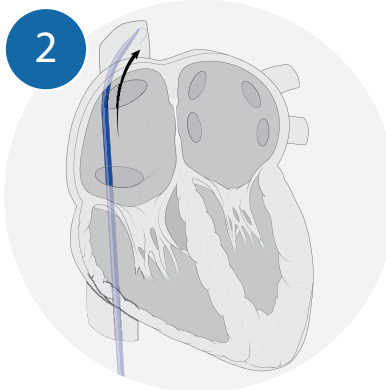


Puncture septum using **NRG**® Transeptal Needle

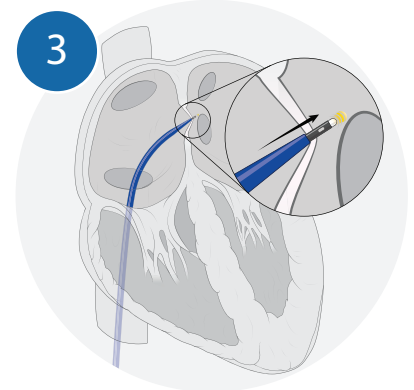


Pre-dilate septum by advancing **ExpanSure**® device, and insert exchange wire

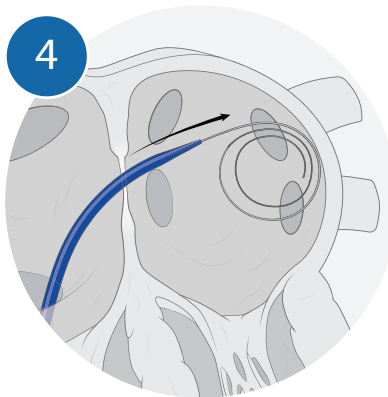

Withdraw **ExpanSure**® device, leaving exchange wire in the left atrium

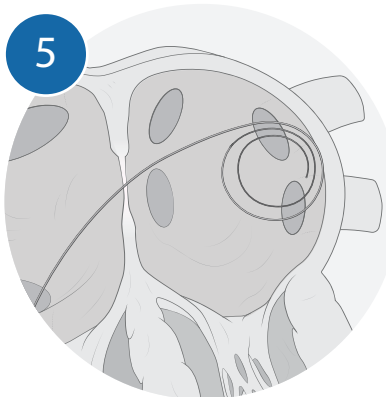

Advance large therapeutic sheath across septum

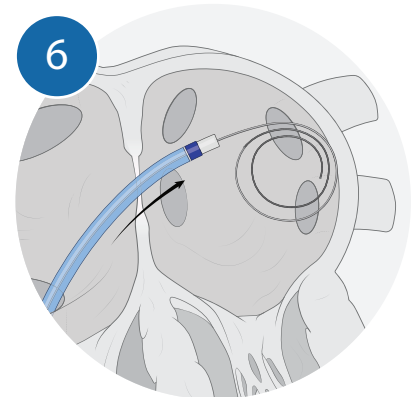

## SPECIFICATIONS

Product number	Outer diameter	Guidewire max outer diameter	Usable length	Compatible needle length
ESD125-35-67-70-N	12.5F	0.035"	67 cm	71 cm
ESD125-35-85-70-N	12.5F	0.035"	85 cm	89 cm

*Packaged with 0.032" outer diameter, 180 cm length guidewire.*

PRM-00390 EN J-1,2 V-4 © Copyright Baylis Medical Company Inc., 2018-2020. Baylis Medical Company Inc. reserves the right to change specifications or to incorporate design changes without notice and without incurring any obligation relating to equipment previously manufactured or delivered. ExpanSure, NRG, 'Seamless access in large sheath cases', and the Baylis Medical logo are trademarks and/or registered trademarks of Baylis Medical Company Inc. in the USA and/or other countries. Patents Pending and/or issued. CAUTION: Federal Law (USA) restricts the use of these devices to or by the order of a physician. Before use, consult product labels and Instructions for Use for Indications for Use, Contraindications, Warnings, Precautions, Adverse Events and Directions for Use. Baylis Medical does not recommend or endorse any specific clinical procedural steps or methodology.

Products shown may not be approved in all jurisdictions.

**Baylis**  
MEDICAL

Baylis Medical Company Inc.  
5959 Trans-Canada Highway  
Montreal, QC Canada H4T 1A1

Tel.: (514) 488-9801 / Fax: (514) 488-7209  
www.baylismedical.com / info@baylismedical.com